

Piccolo[®] Basic Metabolic Panel Plus



For In Vitro Diagnostic Use and Healthcare Professional Use Only

Customer and Technical Service : 1-800-822-2947

Customers outside the US: +49 6155 780 210

AB-PiccoloTechSupport@zoetis.com



Abaxis, Inc.
3240 Whipple Rd.
Union City, CA 94587
USA



Tao of Excellence GmbH
Vorstadt 26,
8200 Schaffhausen
Switzerland



ABAXIS Europe GmbH
Bunsenstr. 9-11
64347 Griesheim
Germany

1. Intended Use

The Piccolo[®] Basic Metabolic Panel Plus, used with the Piccolo Xpress[®] chemistry analyzer, is intended to be used for the *in vitro* quantitative determination of calcium, chloride, creatinine, glucose, lactate dehydrogenase, magnesium, potassium, sodium, total carbon dioxide, and blood urea nitrogen in a clinical laboratory setting or point-of-care location/ near-patient testing. **This disc is for testing heparinized plasma and serum, only.**

2. Summary and Explanation of Tests

The Piccolo Basic Metabolic Panel Plus reagent disc and the Piccolo Xpress chemistry analyzer comprise an *in vitro* diagnostic system that aids the physician in the diagnosis and treatment of the following disorders in adults:

:

Calcium:	General metabolic disease, for example, Hyperparathyroidism, hypothyroidism, bone and chronic renal diseases, tetany.
Chloride:	General metabolic diseases, for example, Dehydration, prolonged diarrhea and vomiting, renal tubular disease, hyperparathyroidism, burns, salt-losing renal diseases, overhydration and thiazide therapy.
Creatinine:	Renal disease and renal dialysis.
Glucose:	Carbohydrate metabolism disorders, for example, adult and juvenile diabetes mellitus and hypoglycemia, hypopituitarism, pancreatitis.
Lactate Dehydrogenase:	Liver diseases, for example, acute viral hepatitis and cirrhosis; and tissue alterations of the liver, and red blood cells.
Magnesium:	Hypomagnesemia and hypermagnesemia.
Potassium:	Renal, endocrine, and metabolic disease as well as iatrogenic causes, for example, Renal glomerular or tubular disease, adrenocortical insufficiency, diabetic ketacidosis, excessive intravenous potassium therapy, sepsis, panhypopituitarism, hyperaldosteronism, malnutrition, hyperinsulinism, metabolic alkalosis and gastrointestinal loss.
Sodium:	Metabolic diseases, for example, Dehydration, diabetes insipidus, loss of hypotonic gastrointestinal fluids, salt poisoning, selective depression of sense of thirst, skin losses, burns, sweating, hyperaldosteronism, CNS disorders, dilutional, depletional and delusional hyponatremia and syndrome of inappropriate ADH secretion.
Total Carbon Dioxide:	Metabolic disorders, for example, metabolic alkalosis and acidosis and primary respiratory alkalosis and acidosis.
Blood Urea Nitrogen:	Renal diseases.

As with any diagnostic test procedure, all other test procedures including the clinical status of the patient, should be considered prior to a final diagnosis.

3. Principle of Method

Calcium (CA)

The first methods used to analyze calcium involved precipitating calcium with an excess of anions.^{1,2,3} Precipitation methods are laborious and often imprecise. The reference method for calcium is atomic absorption spectroscopy; however, this method is not suited for routine use.⁴ Spectrophotometric methods using either *o*-cresolphthalein complexone (CPC) or arsenazo III metallochromic indicators are most commonly used.^{5,6,7} Arsenazo III has a high affinity for calcium and is not temperature dependent as is CPC.


Calcium in the patient sample binds with arsenazo III to form a calcium-dye complex.



The endpoint reaction is monitored at 405 nm, 467 nm and 600 nm. The amount of calcium in the sample is proportional to the absorbance.


Chloride (CL⁻)

The method is based on the determination of chloride-dependent activation of α -amylase activity. Deactivated α -amylase is reactivated by addition of the chloride ion, allowing the calcium to re-associate with the enzyme. The reactivation of α -amylase activity is proportional to the concentration of chloride ions in the sample. The reactivated α -amylase converts the substrate, 2-chloro-*p*-nitrophenyl- α -D-maltotriose (CNP3) to 2-chloro-*p*-nitrophenol (CNP) producing color and α -maltotriose (G3). The reaction is measured bichromatically and the increase in absorbance is directly proportional to the reactivated α -amylase activity and the concentration of chloride ion in the sample.⁸

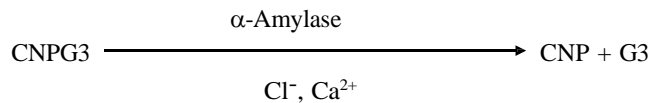

Creatinine (CRE)

The Jaffe method, first introduced in 1886, is still a commonly used method of determining creatinine levels in blood. The current reference method combines the use of Fuller's earth (floridin) with the Jaffe technique to increase the specificity of the reaction.^{9,10} Enzymatic methods have been developed that are more specific for creatinine than the various modifications of the Jaffe technique.^{11,12,13} Methods using the enzyme creatinine amidohydrolase eliminate the problem of ammonium ion interference found in techniques using creatinine iminohydrolase.¹⁴

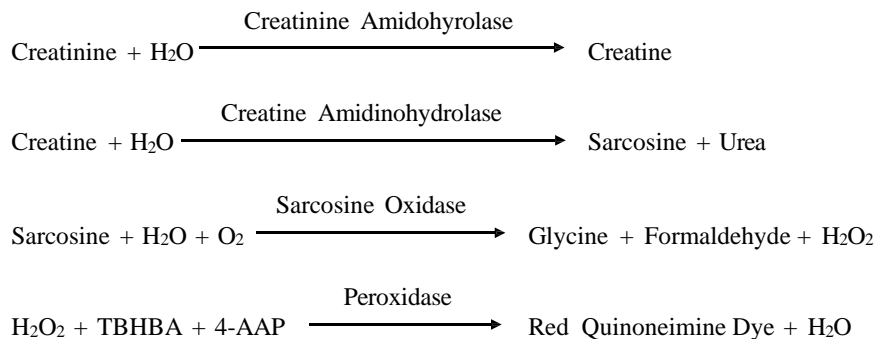

Two cuvettes are used to determine the concentration of creatinine in the sample. Endogenous creatine is measured in the blank cuvette, which is subtracted from the combined endogenous creatine and the creatine formed from the enzyme reactions in the test cuvette. Once the endogenous creatine is eliminated from the calculations, the concentration of creatinine is proportional to the intensity of the red color produced. The endpoint reaction is measured as the difference in absorbance between 550 nm and 600 nm.

eGFR (calculated)


Serum creatinine is routinely measured as an indicator of renal function. Because creatinine is influenced by age, gender and race, chronic kidney disease (CKD) may not be detected using serum creatinine alone. Thus, the National Kidney Disease Education Program strongly recommends that laboratories routinely report an estimated Glomerular Filtration Rate (eGFR) when serum creatinine is measured for patients 18 and older. Routinely reporting the eGFR with all serum creatinine determinations allows laboratories to help identify individuals with reduced kidney function and help facilitate the detection of CKD. Calculated eGFR values of <60 ml/min are generally associated with increased risk of adverse outcomes of CKD.

Calculation of the eGFR is performed by the Piccolo using the patient's age, gender and race. The Piccolo method for creatinine is traceable to the IDMS reference method for creatinine so that the following form of the MDRD equation for calculating the eGFR can be used.

$$\text{GFR (mL/min/1.73 m}^2\text{)} = 175 \times (\text{S}_{\text{cr}})^{-1.154} \times (\text{Age})^{-0.203} \times (0.742 \text{ if female}) \times (1.212 \text{ if African American})$$


Glucose (GLU)

Measurements of glucose concentration were first performed using copper-reduction methods (such as Folin-Wu¹⁵ and Somogyi-Nelson^{16,17}). The lack of specificity in copper-reduction techniques led to the development of quantitative procedures using the enzymes hexokinase and glucose oxidase. The glucose test incorporated into the Piccolo Basic Metabolic Panel Plus reagent disc is a modified version of the hexokinase method, which has been proposed as the basis of the glucose reference method.¹⁸ The reaction of glucose with adenosine triphosphate (ATP), catalyzed by hexokinase (HK), produces glucose-6-phosphate (G-6-P) and adenosine diphosphate (ADP). Glucose-6-phosphate dehydrogenase (G-6-PDH) catalyzes the reaction of G-6-P into 6-phosphogluconate and the reduction of nicotinamide adenine dinucleotide (NAD⁺) to NADH.

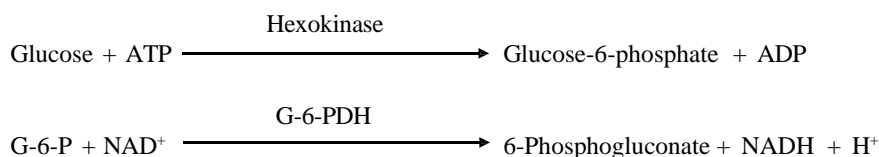

Lactate dehydrogenase (LD)


Lactate dehydrogenase (LD) catalyzes the reversible oxidation of L-lactate to pyruvate with the concurrent reduction of nicotinamide adenine dinucleotide (NAD⁺) to reduced nicotinamide adenine dinucleotide (NADH). The method is based on the lactate-to-pyruvate reaction of Wacker et al.¹⁹ NADH is subsequently oxidized with the simultaneous reduction of p-Iodonitrotetrazolium Violet (INT) to a highly colored formazan dye in a reaction catalyzed by diaphorase.

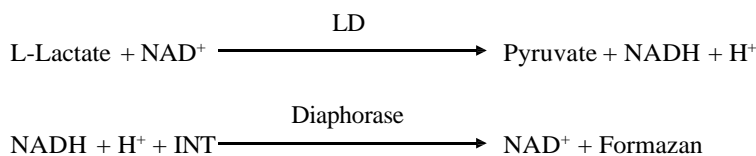

The rate of formation of formazan is measured bichromatically at 500 nm and 630 nm. The rate is directly proportional to the LD activity of the sample.

Magnesium (MG)

The hexokinase (HK) activation method for magnesium is the best-fit method for the Piccolo system in terms of sensitivity, precision, and accuracy.²⁰ The enzymatic magnesium method can be described as:

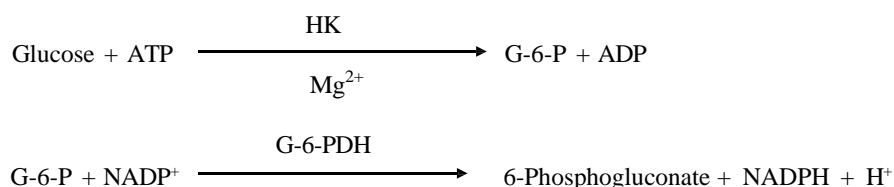


The rate limiting reaction is the HK reaction. Magnesium from the sample activates HK, which in turn catalyzes the breakdown of glucose to form glucose-6-phosphate (G-6-P) and ADP. G-6-P reacts with nicotinamide adenine dinucleotide phosphate (NADP⁺) to form reduced nicotinamide adenine dinucleotide phosphate (NADPH) and 6-phosphogluconate in the presence of glucose-6-phosphate-dehydrogenase (G-6-PDH). This is a first-order rate reaction. The rate of production of NADPH is directly proportional to the amount of magnesium present in the sample. Absorbance is measured bichromatically at 340 nm and 405 nm.

Potassium (K⁺)


Spectrophotometric methods have been developed that allow the measurement of potassium concentration on standard clinical chemistry instrumentation. The Abaxis enzymatic method is based on the activation of pyruvate kinase with potassium and shows excellent linearity and negligible susceptibility to endogenous substances.^{21,22,23} Interference from sodium and ammonium ion are minimized with the addition of Kryptofix and glutamate dehydrogenase, respectively.²³

In the coupled-enzyme reaction, pyruvate kinase (PK) dephosphorylates phosphoenolpyruvate (PEP) to form pyruvate. Lactate dehydrogenase (LD) catalyzes conversion of pyruvate to lactate. Concomitantly, NADH is oxidized to NAD⁺. The rate of change in absorbance difference between 340 nm and 405 nm due to the conversion of NADH to NAD⁺ is directly proportional to the amount of potassium in the sample.

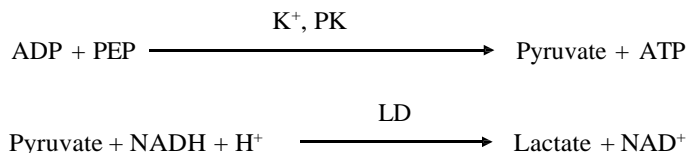

Sodium (NA⁺)


Colorimetric and enzymatic methods have been developed that allow the measurement of sodium concentration on standard clinical chemistry instrumentation.^{24,25,26} In the Abaxis enzymatic reaction, β-galactosidase is activated by the sodium in the sample. The activated enzyme catalyzes the reaction of o-nitrophenyl-β-D-galactopyranoside (ONPG) to o-nitrophenol and galactose.

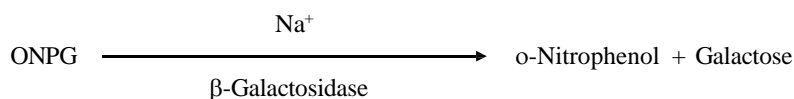

Total Carbon Dioxide (tCO₂)

Total carbon dioxide in serum or plasma exists as dissolved carbon dioxide, carbamino derivatives of proteins, bicarbonate and carbonate ions and carbonic acid. Total carbon dioxide can be measured by pH indicator, CO₂ electrode and spectrophotometric enzymatic methods, which all produce accurate and precise results.^{27,28} The enzymatic method is well suited for use on a routine blood chemistry analyzer without adding complexity.


In the enzymatic method, the specimen is first made alkaline to convert all forms of carbon dioxide (CO₂) toward bicarbonate (HCO₃⁻). Phosphoenolpyruvate (PEP) and HCO₃⁻ then react to form oxaloacetate and phosphate in the presence of phosphoenolpyruvate carboxylase (PEPC). Malate dehydrogenase (MDH) catalyzes the reaction of oxaloacetate and reduced nicotinamide adenine dinucleotide (NADH) to NAD⁺ and malate. The rate of change in absorbance due to the conversion of NADH to NAD⁺ is directly proportional to the amount of the CO₂ in the sample.

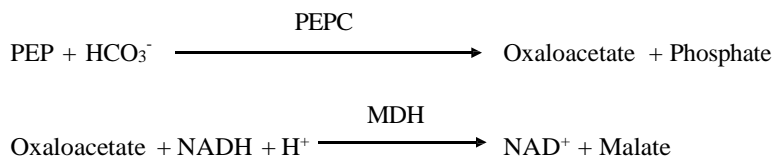

Blood Urea Nitrogen (BUN)

Urea can be measured both directly and indirectly. The diacetyl monoxime reaction, the only direct method to measure urea, is commonly used but employs dangerous reagents.²⁹ Indirect methods measure ammonia created from the urea; the use of the enzyme urease has increased the specificity of these tests.³⁰ The ammonia is quantitated by a variety of methods, including nesslerization (acid titration), the Berthelot technique^{31,32} and coupled enzymatic reactions.^{33,34} Catalyzed Berthelot procedures, however, are erratic when measuring ammonia.³⁵ Coupled-enzyme reactions are rapid, have a high specificity for ammonia, and are commonly used. One such reaction has been proposed as a candidate reference method.³⁶

In the coupled-enzyme reaction, urease hydrolyzes urea into ammonia and carbon dioxide. Upon combining ammonia with α-ketoglutarate and NADH, the enzyme glutamate dehydrogenase (GLDH) oxidizes NADH to NAD⁺.

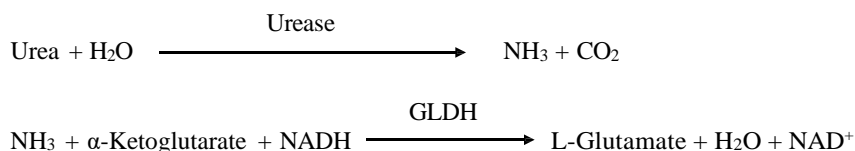

The rate of change of the absorbance difference between 340 nm and 405 nm is caused by the conversion of NADH to NAD⁺ and is directly proportional to the amount of urea present in the sample.

4. Principle of Operation

Refer to the Piccolo Xpress chemistry analyzer Operator's Manual, for the Principles and Limitations of the Procedure.

5. Description of Reagents

Reagents

Each Piccolo Basic Metabolic Panel Plus reagent disc contains dry test-specific reagent beads (described below). A dry sample blank reagent (comprised of buffer, surfactants, excipients, and preservatives) is included in each disc for use in calculating concentrations of calcium, chloride, glucose, lactate dehydrogenase, magnesium, potassium, sodium, total carbon dioxide, and blood urea nitrogen. A dedicated sample blank is included in the disc for creatinine (CRE). Each disc also contains a diluent consisting of surfactants and preservatives.

Table 1: Reagents

Component	Quantity/Disc
2, 4, 6-Tribromo-3-hydroxybenzoic acid	188 µg
2-Chloro-4-nitrophenyl -alpha-maltotrioxide (CNPG3)	105 µg
4,7,13,16,21-Pentaoxa-1,10-diazabicyclo[8.8.5]triscosane	86 µg
4-Aminoantipyrine*HCl	32 µg
N-Acetyl cysteine	15.3 µg
Adenosine 5'-diphosphate	36 µg
Adenosine 5'-diphosphate, trilithium salt	3.7 µg
Adenosine 5'-triphosphate, disodium salt	28 µg
4-Aminoantipyrine	5.4 µg
Amylase	0.2856 U
Arsenazo III, sodium salt	3.5 µg
Arsenazo III, disodium salt	3.5 µg
Ascorbate oxidase	0.3 U
Bovuminar reagent, pure powder	815 µg
Calcium acetate	50.4 µg
Citric acid, trisodium salt	1134 µg
Creatine amidinohydrolase	3 U
Creatinine amidohydrolase	1 U
Dextran, Low Fraction	1780 µg
Diaphorase	0.084 U
3,5-Dichloro-2-hydroxybenzenesulfonic acid , Sodium salt (DHBSA)	34.8 µg
Ethylene glycol-bis(β-aminoethyl ether)-N,N',N'-tetraacetic acid (EGTA)	18.5 µg
Ethylenediaminetetraacetic acid (EDTA)	417.59 µg
Ethylenediaminetetraacetic acid (EDTA), disodium salt	357 µg
Ethylenediaminetetraacetic acid (EDTA), Tetrasodium Salt	180 µg
β-Galactosidase	0.005 U
Glucose	64 µg
Glucose-6-phosphate dehydrogenase	0.068 U
Glutamate dehydrogenase	0 U
Hexokinase	0.071 U
L-glutamic acid	9 µg
Glutamine synthetase	0.2 U
Lactate dehydrogenase	0 U
Lactate dehydrogenase (LDH)	0.1 U
p-Iodonitrotetrazolium violet (INT)	5.082 µg
Imidazole	57 µg

<i>myo</i> -Inositol	201 µg
α-Ketoglutaric acid	18 µg
Lactate dehydrogenase	0.1 U
Lithium lactate	96.77 µg
Malate dehydrogenase	0.1 U
Magnesium acetate, tetrahydrate	22 µg
Magnesium sulfate, heptahydrate	122 µg
Manganese chloride	10 µg
Nicotinamide adenine dinucleotide, sodium salt	69 µg
D-Mannitol	3019 µg
β-Nicotinamide adenine dinucleotide (NAD ⁺)	88.8 µg
β-Nicotinamide adenine dinucleotide, reduced (NADH)	33 µg
β-Nicotinamide adenine dinucleotide phosphate (NADP ⁺), sodium salt	29.6 µg
<i>o</i> -Nitrophenyl-β-D-galactopyranoside (ONPG)	22 µg
<i>n</i> -Octylglucoside	21 µg
Peroxidase	1 U
Phosphoenolpyruvate	32 µg
Phosphoenolpyruvate carboxylase 0.001Phosphoenolpyruvate carboxylase	0.002 U
Polyethylene glycol, 3400	612 µg
Polyvinylpyrrolidone	6 µg
Potassium ferrocyanide	0.6 µg
Pyruvate kinase	0.01 U
Sarcosine oxidase	1 U
Sodium chloride	673 µg
D(+) Trehalose, dihydrate	861 µg
Triethanolamine-hydrochloride	138.72 µg
Tris(hydroxymethyl)aminomethane	585.4 µg
Tris(hydroxymethyl)aminomethane*HCl	40.91 µg
Triton X-100	241.22 µg
Urease	0.05 U
Uricase	0 U
Buffers, surfactants, excipients and preservatives	

Warnings and Precautions

- For *In vitro* Diagnostic Use
- The diluent container in the reagent disc is automatically opened when the analyzer drawer closes. A disc with an opened diluent container cannot be re-used. Ensure that the sample or control has been placed into the disc before closing the drawer.
- Used reagent discs contain human body fluids. Follow good laboratory safety practices when handling and disposing of used discs.³⁸ See the Piccolo Xpress Chemistry Analyzer or Piccolo Xpress chemistry analyzer Operator's Manual for instructions on cleaning biohazardous spills.
- The reagent discs are plastic and may crack or chip if dropped. Never use a dropped disc as it may spray biohazardous material throughout the interior of the analyzer.
- Reagent beads may contain acids or caustic substances. The operator does not come into contact with the reagent beads when following the recommended procedures. In the event that the beads are handled (e.g., cleaning up after dropping and cracking a reagent disc), avoid ingestion, skin contact, or inhalation of the reagent beads.
- This mixture does not contain substances assessed to be vPvB / PBT according to Regulation (EC) No 1907/2006, Annex XIII. This product contains components considered to have endocrine disrupting properties for environment, according to REACH Article 57(f), Regulation (EU) 2018/605 or Regulation (EU) 2017/2100. Refer to the Safety Data Sheet (SDS) for specific details.

- Materials incorporated into the device may consist of substances derived from other microbial and animal sourced materials.

Instructions for Reagent Handling

Reagent discs may be used directly from the refrigerator without warming. Do not allow discs sealed in their foil pouches to remain at room temperature at 20-25°C (68-77°F) longer than 48 hours prior to use. Open the sealed foil pouch, remove the disc and use according to the instructions provided in the Piccolo Xpress chemistry analyzer Operator's Manual. A disc not used within 20 minutes of opening the pouch should be discarded.

Storage

Store reagent discs in their sealed pouches at 2-8 °C (36-46 °F). Do not expose opened or unopened discs to direct sunlight or temperatures above 32 °C (90 °F). Reagent discs may be used until the expiration date included on the package. The expiration date is also encoded in the bar code printed on the bar code ring. An error message will appear on the Piccolo Xpress chemistry analyzer display if the reagents have expired.

Indications of Reagent Disc Instability/Deterioration

A torn or otherwise damaged pouch may allow moisture to reach the unused disc and adversely affect reagent performance. Do not use a disc from a damaged pouch.

6. Instrument

See the Piccolo Xpress chemistry analyzer Operator's Manual for complete information on use of the analyzer.

7. Sample Collection and Preparation

Sample collection techniques are described in the "Sample Collection" section of the Piccolo Xpress chemistry analyzer Operator's Manual.

- The minimum required sample size is ~100 µL of heparinized plasma, serum, or control material. The reagent disc sample chamber can contain up to 120 µL of sample.
- Do not shake the collection tube; shaking may cause hemolysis. Hemolysis will cause erroneously high results in **potassium** and **lactate dehydrogenase** assays.
- Hemolysis may cause erroneously high results in **potassium** assays. This problem may go undetected when analyzing whole blood (release of potassium from as few as 0.5% of the erythrocytes can increase the potassium serum level by 0.5 mmol/L). In addition, even unhemolyzed specimens that are not promptly processed may have increased potassium levels due to intracellular potassium leakage.⁴⁰
- In addition, even unhemolyzed specimens that are not promptly processed may have increased potassium levels due to intracellular potassium leakage.³⁹
- Use only lithium heparin evacuated specimen collection tubes for plasma samples. Use no-additive evacuated specimen collection tubes or serum separator tubes for serum samples.
- Start the test within 10 minutes of transferring the sample into the reagent disc.
- The concentration of **total carbon dioxide** is most accurately determined when the assay is done immediately after opening the tube and as promptly as possible after collection and processing of the blood in the unopened tube. Ambient air contains far less carbon dioxide than does plasma, and gaseous dissolved carbon dioxide will escape from the specimen into the air, with a consequent decrease in carbon dioxide value of up to 6 mmol/L in the course of 1 hour.⁴⁰
- Refrigerating whole blood samples can cause significant changes in concentration of **creatinine** and **glucose**³⁷. The sample may be separated into plasma or serum and stored in capped sample tubes at 2-8°C (36-46°F) if the sample cannot be run within 60 minutes.
- Whole blood venipuncture samples should be run within 60 minutes of collection.^{52,55} **Glucose** concentrations are affected by the length of time since the patient has eaten and by the type of sample collected from the patient. To accurately determine glucose results, samples should be obtained from a patient who has been fasting for at least 12 hours. The glucose concentration decreases approximately 5-12 mg/dL in 1 hour in uncentrifuged samples stored at room temperature.⁵³

- Whole blood samples should only be obtained via venipuncture, not from capillary blood.

8. Procedure

Materials Provided

- One Piccolo Basic Metabolic Panel Plus PN : 400-1031 (a box of discs PN : 400-0031)

Materials Required but not Provided

- Piccolo Xpress chemistry analyzer
- Sample transfer pipettes (fixed volume approximately 100 µL) and tips are provided with each Piccolo Xpress chemistry analyzer and may be reordered from Abaxis.
- Commercially available control reagents recommended by Abaxis (refer to Piccolo Xpress chemistry analyzer Operator’s Manual)
- Timer

Test Parameters

The Piccolo Xpress chemistry analyzer operates at ambient temperatures between 15°C and 32 °C (59-90 °F). The analysis time for each Piccolo Basic Metabolic Panel Plus is less than 14 minutes. The analyzer maintains the reagent disc at a temperature of 37 °C (98.6 °F) over the measurement interval.

Test Procedure

The complete sample collection and step-by-step operating procedures are detailed in the Piccolo Xpress chemistry analyzer Operator’s Manual.

Calibration

The Piccolo Xpress chemistry analyzer is calibrated by the manufacturer before shipment. The bar code printed on the bar code ring provides the analyzer with disc-specific calibration data. See the Piccolo Xpress chemistry analyzer Operator’s Manual.

The Piccolo Xpress chemistry analyzer is calibrated using internal calibrators or reference materials. The methods and materials used in the procedures for the calibration and control value assignments are traceable either to the comparative method and/or to the standards listed below:

- IFCC – International Federation of Clinical Chemistry
- NIST – National Institute of Standards and Technology
- CRMLN – Cholesterol Reference Method Laboratory Network
- SRM – Standard Reference Material
- CRM – Certified Reference Material

The calibration and value assignment processes are in compliance with EN ISO 17511, Metrological Traceability of Values Assigned to Calibrators and Control Materials.

The measured analytes in the Basic Metabolic Panel Plus are traceable to the following reference materials or methods. The Piccolo Xpress Analysis System controls and calibration verification materials are validated for use only with the Piccolo Xpress Analysis System and assigned values may not be commutable with other methods. For specific analyte reference standard methods, refer to Section 3. Principles of Method.

Analyte	Calibration Standard	Method
CA	Correlation to Beckman LX-20 / DX-20	Arsenazo III Dye
CL	Correlation to Beckman LX-20 / DX-20	Enzymatic
CRE	NIST SRM #967	Enzymatic
BUN	NIST SRM #912A	Enzymatic
GLU	NIST SRM #909	Enzymatic
LDH	Correlation to Beckman LX-20 / DX-20	Enzymatic

Mg	NIST SRM #909B	Enzymatic
K+	NIST SRM #909	Enzymatic
NA+	Correlation to Beckman LX-20 / DX-20	Enzymatic
tCO ₂	Correlation to Beckman LX-20 / DX-20	Enzymatic

Quality Control

See Section 6 (Calibration and Quality Control) of the Piccolo Xpress chemistry analyzer Operator's Manual. Performance of the Piccolo Xpress chemistry analyzer can be verified by running controls. For a list of approved quality control materials with acceptance ranges, please contact Abaxis Technical Support. Other human serum or plasma-based controls may not be compatible. Quality control materials should be stored as per the package-insert included with the controls.

If control results are out of range, repeat one time. If still out of range, call Abaxis Technical Support. Do not report results if controls are outside their labeled limits. See Piccolo Xpress chemistry analyzer Operator's Manual for a detailed discussion on running, recording, interpreting, and plotting control results.

Waived Laboratories (US only): Abaxis recommends control testing as follows:

- at least every 30 days
- whenever the laboratory conditions have changed significantly, e.g. Piccolo moved to a new location or changes in temperature control
- when training or retraining of personnel is indicated
- with each new lot (CLIA waived tests in waived status labs)

Non-Waived Laboratories (US only): Abaxis recommends control testing to follow federal, state, and local guidelines.

And the non-waived ones will use the below:

- Performance of the Piccolo Xpress chemistry analyzer can be verified by running controls. For a list of approved quality control materials with acceptance ranges, please contact Abaxis Technical Support. Other human serum or plasma-based controls may not be compatible. Quality control materials should be stored as per the package-insert included with the controls.
- If control results are out of range, repeat one time. If still out of range, call Abaxis Technical Support. Do not report results if controls are outside their labeled limits. See the Piccolo Xpress chemistry analyzer Operator's Manual for a detailed discussion on running, recording, interpreting, and plotting control results.

9. Results

The Piccolo Xpress chemistry analyzer automatically calculates and prints the analyte concentrations in the sample. Details of the endpoint and rate reaction calculations are found in the Piccolo Xpress chemistry analyzer Operator's Manual.

Interpretation of results is detailed in the Operator's Manual. Results are printed onto result cards or paper rolls supplied by Abaxis. The result cards or paper rolls have an adhesive backing for easy placement in the patient's files.

The reaction for each analyte occurs at 37°C (98.6°F).

10. Limitations of Procedure

General procedural limitations are discussed in the Piccolo Xpress chemistry analyzer Operator's Manual.

- Only heparinized plasma or serum may be used with this disc due to the susceptibility of falsely high LD values from ruptured blood cells.
- The only anticoagulant **recommended for use** with the Piccolo Xpress chemistry analyzer is **lithium heparin**. Do not use sodium heparin. Abaxis has performed studies demonstrating that EDTA, fluoride, oxalate, and any anticoagulant containing ammonium ions will interfere with at least one chemistry contained in the Piccolo Basic Metabolic Panel Plus reagent disc.

- Samples with hematocrits in excess of 62% packed red cell volume (a volume fraction of 0.62) may give inaccurate results. Samples with high hematocrits may be reported as hemolyzed. These samples may be spun down to get plasma then re-run in a new reagent disc.
- **Any result for a particular test that exceeds the assay range should be analyzed by another approved test method or sent to a referral laboratory. Do not dilute the sample and run it again on the Piccolo Xpress chemistry analyzer.**

Warning : Extensive testing of the Piccolo Xpress chemistry analyzer has shown that, in very rare instances, sample dispensed into the reagent disc may not flow smoothly into the sample chamber. Due to the uneven flow, an inadequate quantity of sample may be analyzed and several results may fall outside the reference ranges. The sample may be re-run using a new reagent disc.

Note: Operators shall report any serious incident that has occurred in relation to the device to the manufacturer.

Interference

Substances were tested as interferents with the analytes. Human serum pools were prepared. The concentration at which each potential interferent was tested was based on the testing levels in CLSI EP7-A.⁴¹

Effects of Endogenous Substances

- Physiological interferents (hemolysis, icterus and lipemia) cause changes in the reported concentrations of some analytes. The sample indices are printed on the bottom of each printout to inform the operator about the levels of interferents present in each sample.
- The Piccolo Xpress chemistry analyzer suppresses any results that are affected by >10% interference from hemolysis, lipemia or icterus. “HEM”, “LIP”, or “ICT” respectively, is printed on the printout in place of the result, with the exception of LD. See an explanation of the effects on LD in the next bullet point. For the endogenous limits, please contact Abaxis Technical Service.
- Significant levels of LD are found in blood cells. Rupture of these cells can lead to increased levels of LD. Hence; all LD assays are sensitive to hemolysis due to release of LD from the red blood cells. There was no significant interference in LD (> 10%) when HEM values up to 50 mg/dL were tested. For the LD assay, only, if HEM is greater than 50 and less than or equal to 100 mg/dL, the LD value will be printed followed by an “H” indicating some additional influence from hemolysis. If the HEM is greater than 100 mg/dL and less than or equal to 150 mg/dL, the LD value will be preceded by “<” and followed by an “H.” Thus, indicating that the true LD recovery is less than that reported. The purpose of these annotations is to help interpret LD activity in the presence of small amounts of hemolysis. For values of HEM above 150, no result will be indicated and only “HEM” will be printed.

Extremely elevated amylase levels (>9,000 U/L) will have a significant effect, >10% increase, on the chloride result. The concentration of amylase is not evaluated by the Piccolo Xpress chemistry analyzer for each specimen.

- The potassium assay in the Piccolo Xpress chemistry analyzer is a coupled pyruvate kinase (PK) / lactate dehydrogenase (LD or LDH) assay. Therefore, in cases of extreme muscle trauma or highly elevated levels of creatine kinase (CK), the Piccolo may recover a falsely elevated potassium (K+) value. In such cases, unexpected high potassium recoveries need to be confirmed utilizing a different methodology.
- For maximum levels of endogenous substances contact Abaxis Technical Support.

Effects of Exogenous and Therapeutic Substances

Thirty-eight drugs and metabolites were selected as potential interferents with chloride, calcium, creatinine, glucose, magnesium, potassium, sodium, total carbon dioxide and blood urea nitrogen methods. The drugs and metabolites that were evaluated were chosen based on recommendations by Young.⁴² Eleven of these were tested by the LD assay and are marked with *. Two additional substances (lactic acid and lithium citrate), marked with (LD only), were selected as potential interferents for the LD assay and tested with it alone. Significant interference is defined as a >10% shift in the result from control sample. Human serum pools were supplemented with a known concentration of the drugs or chemicals and then analyzed. **Please see Table 2 for a list of exogenous and therapeutic substances evaluated. Please see Table 3 for a list of analytes where interference was observed.**

Table 2 : Therapeutic Substances Evaluated

	Highest Concentration Tested (mg/dL)
Acetaminophen*	100
Acetoacetate*	102
Acetylsalicylic acid*	50
Ampicillin	30
Ascorbic acid	20
Ascorbic acid* (LD)	3
Caffeine*	10
Cephalothin (Keflin)	400
Chloramphenicol	100
Cimetidine	16
Dopamine	19
Epinephrine	1
Erythromycin	10
Glutathione	30
Hydrochlorothiazide	7.5
Ibuprofen*	50
Isoniazide	4
Ketoprofen	50
L-Dopa	5
Glutathione	30
Lactic acid (LD only)	60
Lidocaine*	1
Lithium citrate (LD only)	3.5
Lithium Lactate	84
Methicillin	100
Methotrexate	0.5
Methotrexate* (LD)	450
Metronidazole	5
Nafcillin	1
Nitrofurantoin	20
Oxacillin	1
Oxaloacetate*	132
Penicillin G	100
Phenytoin (5,5-Diphenylhydantion)*	3
Proline	4
Pyruvate*	44
Rifampin	0.5
Salicylic Acid	50
Sulfasalazine	150
Sulfanilamide	50
Theophylline	20

^A Updated Methotrexate therapeutic concentration based on CLSI Vol. 22 No. 27 Guideline.

Table 3: Substances With Significant Interference >10%

	Concentration Which Produces > 10% Interference
Calcium	None
Chloride	None
Creatinine	
Ascorbic Acid	20
Dopamine	19
L-Dopa	5
Epinephrine	1
Glutathione	30
Glucose	
Oxaloacetate	132
Pyruvate	44
Lactate dehydrogenase	
Oxaloacetate	99 (no effect at 66)
Magnesium	None
Potassium	
Penicillin G	100
Sulfasalazine	150
Sodium	
Cephalothin	400
Methotrexate	0.5
Penicillin G	100
Total Carbon Dioxide	
Acetaminophen	100
Ascorbic Acid	20
Cephalothin	400
Cimetidine	16
Erythromycin	10
Lidocaine	1
Methotrexate	0.5
Nitrofurantoin	20
Salicylic Acid	50
Sulfasalazine	150
Blood Urea Nitrogen	None

^BUpdated Methotrexate therapeutic concentration based on CLSI Vol. 22 No. 27

Table 4 : Concentration of Analytes in Serum Pool Used for Interference Studies

Analyte	Concentration
Calcium	9.5 mg/dL
Chloride	93 mmol/L
Creatinine	4.1 mg/dL
Glucose	96 mg/dL
Lactate Dehydrogenase	276 U/L and 703 U/L
Magnesium	4.3 mg/dL
Potassium	3.8 mmol/L
Sodium	124 mmol/L
Total Carbon Dioxide	6 mmol/L
Blood Urea Nitrogen	26 mg/dL

- For the Chloride assay, bromide at toxic levels (≥ 15 mmol/L) can cause a significant effect ($> 10\%$ increase), on the chloride result. Iodide at very high concentrations (30 mmol/L, highest level tested) has no effect. Normal physiological levels of bromide and iodide do not interfere with the Piccolo Chloride Test System.

11. Expected Values

Samples from 60 - 150 adult males and females were analyzed on the Piccolo Xpress chemistry analyzer to determine the reference interval for the analytes, with the exception of LD. For LD, the reference interval was established by applying linear regression statistics from a correlation study versus the Beckman Synchron LX20 to the LX20's published reference interval. These intervals are provided as a guideline only. It is recommended that your office or institution establish normal ranges for your particular patient population.⁴⁶

Table 5 : Piccolo Reference Intervals**

Analyte	Common Units	SI Units
Calcium	8.0 – 10.3 mg/dL	2.0 – 2.58 mmol/L
Chloride	98 – 108 mmol/L	98 – 108 mmol/L
Creatinine	0.6 – 1.2 mg/dL	53 – 106 μ mol/L
Glucose	73 – 118 mg/dL	4.05 – 6.55 mmol/L
Lactate Dehydrogenase*	99 – 192 U/L	99 – 192 U/L
Magnesium	1.6 – 2.3 mg/dL	0.66 – 0.95 mmol/L
Potassium	3.6 – 5.1 mmol/L	3.6 – 5.1 mmol/L
Sodium	128 – 145 mmol/L	128 – 145 mmol/L
Total Carbon Dioxide	18 – 33 mmol/L	18 – 33 mmol/L
Blood Urea Nitrogen (BUN)	7 – 22 mg/dL	2.5 – 7.9 mmol urea/L

* A small increase (approximately 6 U/L) in lactate dehydrogenase was observed in serum when compared to heparinized plasma. This increase is consistent with the difference between serum and plasma for LD as described in the literature.^{47,48} LD is released from blood cells during the coagulation process in the preparation of serum.

**Data was generated using equivalence device Piccolo Blood Chemistry Analyzer.

12. Performance Characteristics

Linearity

The chemistry for each analyte is linear over the dynamic range listed below when the Piccolo Xpress chemistry analyzer is operated according to the recommended procedure (refer to the Piccolo Xpress chemistry analyzer Operator's Manual).

Table 6 : Piccolo Dynamic Ranges

Analyte	Common Units	SI Units
Calcium	4.0 – 16.0 mg/dL	1.0 – 4.0 mmol/L
Chloride	80 – 135 mmol/L	80 – 135 mmol/L
Creatinine	0.2 – 20 mg/dL	18 – 1768 µmol/L
Glucose	10 – 700 mg/dL	0.56 – 38.9 mmol/L
Lactate Dehydrogenase	50 – 1000 U/L	50 – 1000 U/L
Magnesium	0.1 – 8.0 mg/dL	0.04 – 3.3 mmol/L
Potassium	1.5 – 8.5 mmol/L	1.5 – 8.5 mmol/L
Sodium	110 – 170 mmol/L	110 – 170 mmol/L
Total Carbon Dioxide	5 – 40 mmol/L	5 – 40 mmol/L
Blood Urea Nitrogen (BUN)	2 – 180 mg/dL	0.7 – 64.3 mmol urea/L

If the analyte concentration is above the measuring range (dynamic range), but less than the system range, the printout will indicate a “>” sign at the upper limit and an asterisk after the number, e.g. ALT >2000* U/L. If lower than the dynamic range, a “<” will be printed with an asterisk, e.g. ALT <5* U/L. For values that are grossly beyond the measurement range (system range), “~~~” will be printed instead of a result. Any time “~~~” appears on a printout, collect a new sample and rerun the test. If results for the second sample are suppressed again, please call Abaxis Customer Service.

Sensitivity (Limits of Detection)

The lower limit of the reportable (dynamic) range for each analyte is : calcium 4.0 mg/dL (1.0 mmol/L); chloride 80 mmol/L; creatinine 0.2 mg/dL (18 µmol/L); glucose 10 mg/dL (0.56 mmol/L); lactate dehydrogenase 50.0 U/L; magnesium 0.1 mg/dL (0.04 mmol/L); potassium 1.5 mmol/L; sodium 110 mmol/L; total carbon dioxide 5 mmol/L; and blood urea nitrogen 2.0 mg/dL (0.7 mmol urea/L).

Precision

Precision studies were conducted using CLSI EP5-A2 guidelines⁴⁹ with modifications based on CLSI EP18-A⁵⁰ for unit-use devices. Results for within-run and total precision were determined using two levels of commercially available control materials and in the case of potassium two levels of plasma pools. The studies made use of multiple instruments. Calcium, creatinine, glucose, sodium and blood urea nitrogen testing was performed at one site; potassium and total carbon dioxide testing was performed at two sites over 20 days; chloride, lactate dehydrogenase and magnesium testing was done at two sites over a period of five days. Potassium testing was conducted at a CLIA waived site making use of three analyzers, one lot of reagent discs, and two operators over five days.

Results of precision studies are shown in Table 7.

Table 7 : Precision

Analyte	Sample Size	Within-Run	Total
Calcium (mg/dL)			
Control 1	N = 80		
Mean		8.6	8.6
SD		0.21	0.25
%CV		2.4	2.9
<u>Control 2</u>			
Mean		11.8	11.8
SD		0.39	0.40
%CV		3.3	3.4
Chloride (mmol/L)			
Control 1	N = 160		
Mean		97.8	97.8
SD		1.63	1.74
%CV		1.7	1.7
Control 2			
Mean		113.6	113.6
SD		1.97	2.22
%CV		1.7	2.0

Creatinine (mg/dL)	N=80		
Control 1			
Mean		1.1	1.1
SD		0.14	0.14
%CV		12.5	13.1
Control 2			
Mean		5.2	5.2
SD		0.23	0.27
%CV		4.4	5.2
Glucose (mg/dL)	N=80		
Control 1			
Mean		66	66
SD		0.76	1.03
%CV		1.1	1.6
Control 2			
Mean		278	278
SD		2.47	3.84
%CV		0.9	1.4
Lactate Dehydrogenase (U/L)	N=80		
Control 1			
Mean		87	87
SD		3.0	4.4
%CV		3.4	5.0
Control 2			
Mean		350	350
SD		3.8	7.0
%CV		1.1	2.0
Magnesium (mg/dL)	N =80		
<u>Control 1</u>			
Mean		1.9	1.9
SD		0.03	0.06
%CV		1.7	3.4
<u>Control 2</u>			
Mean		3.9	3.9
SD		0.04	0.10
%CV		1.0	2.6
Potassium (mmol/L)	N = 130		
<u>Control 1</u>			
Mean		3.2	3.2
SD		0.09	0.11
%CV		2.8	3.3
<u>Control 2</u>	N = 149		
Mean		6.2	6.2
SD		0.09	0.10
%CV		1.4	1.7
<u>Plasma Pool 1</u>	N = 150		
Mean		3.2	3.2
SD		0.07	0.09
CV		2.3	2.9
<u>Plasma Pool 2</u>	N = 150		
Mean		5.4	5.4
SD		0.09	0.10
CV		1.6	1.9
Sodium (mmol/L.)	N = 80		

<u>Control 1</u>			
Mean		143.5	143.5
SD		2.28	2.28
%CV		1.6	1.6
<u>Control 2</u>			
Mean		120.0	120.0
SD		2.13	2.13
%CV		1.8	1.8
Total Carbon Dioxide	N = 120		
(mmol/L)			
<u>Control 1</u>			
Mean		21.4	21.4
SD		2.29	2.29
%CV		10.7	10.7
<u>Control 2</u>			
Mean		10.5	10.5
SD		0.90	0.90
%CV		8.6	8.6
Blood Urea Nitrogen (mg/dL)	N = 80		
<u>Control 1</u>			
Mean		19	19
SD		0.35	0.40
%CV		1.9	2.1
<u>Control 2</u>			
Mean		65	65
SD		1.06	1.18
%CV		1.6	1.8

Whole Blood Precision for Potassium

Whole blood precision was tested at a CLIA waived site by two CLIA waiver operators. The study used four Piccolo Xpress Analyzers with 16 replicates per sample for four (4) fresh, lithium heparin whole blood samples.

Table 7: Whole Blood Precision for Potassium

Potassium (mmol/L)	Sample Size	Within-Run	Total
Whole Blood 1	N = 16		
Mean		3.9	3.9
SD		0.06	0.11
CV		1.6	2.8
Whole Blood 2	N = 16		
Mean		4.0	4.0
SD		0.11	0.14
CV		2.9	3.4
Whole Blood 3	N = 16		
Mean		4.0	4.0
SD		0.11	0.15
CV		2.8	3.9
Whole Blood 4	N = 16		
Mean		4.0	4.0
SD		0.11	0.13

Correlation

Heparinized serum samples and plasma samples for LD assay were collected and assayed by a comparative method(s). In some cases, high and low supplemented samples were used to cover the dynamic range. The samples were chosen to meet the distribution values in CLSI EP9-A2 guideline.⁵¹ Representative correlation statistics are shown in Table 8.

Table 8: Correlation with Comparative Method(s)**

	Correlation Coefficient	Slope	Intercept	SEE	N	Sample Range	Comparative Method
Calcium (mg/dL)	0.991*	0.990	-0.4	0.17	25	5.2-11.9	Paramax®
	0.673	0.742	1.8	0.22	81	8.1-9.9	Beckman
Chloride (mmol/L)	0.978	0.982	-1.1	1.84	120	71 - 118	Vitros 950® Ortho
Creatinine (mg/dL)	0.993	0.926	0.0	0.15	260	0.4-14.7	Paramax®
	0.987	0.866	0.1	0.16	107	0.4-7.5	Beckman
Glucose (mg/dL)	0.987	1.009	-2.8	3.89	251	72-422	Paramax®
	0.997	0.943	1.2	4.69	91	56-646	Beckman
Lactate Dehydrogenase (U/L)	0.994	0.983	3.8	26.3	60	44 – 1172	Synchron® LX20 Beckman
Magnesium (mg/dL)	0.992	0.990	0.0	0.16	44	0.8 - 6.8	Inductively Coupled Plasma-Atomic Optical Emission Spectroscopy (ICP-OES)
Potassium (mmol/L) Whole Blood (waived laboratory)	0.984	0.99	0.13	0.10	130	1.3-9.5	Siemens VISTA Plasma
Potassium (mmol/L) Whole Blood (moderately complex laboratory)	0.984	0.98	0.12	0.18	178	1.5-8.6	Siemens VISTA Plasma
Potassium (mmol/L) Serum (moderately complex laboratory)	0.99	0.98	0.06	0.14	178	1.4-8.5	Siemens VISTA Serum
Total Carbon Dioxide (mmol/L)	0.947	0.903	2.0	0.84	60	6-39	Cobra Fara® Roche
Blood Urea Nitrogen (mg/dL)	0.964	0.923	0.5	1.08	251	6-52	Paramax® Beckman
	0.983	0.946	0.0	0.66	92	6-38	

* Serum samples from hospitalized patients provided a broader, and possibly more useful, sample range than did venous whole blood samples from out-patients. Correlation statistics for the Piccolo calcium test are from these serum samples.

**Data was generated using equivalence device Piccolo Blood Chemistry Analyzer.

It should be noted that serum will typically give higher results for K+ compared to whole blood or plasma for physiological reasons. The variation can range from approximately 0.2 to 0.9mmol/L and is dependent on a number of factors. The primary effect is dependent upon the number of blood cells present in the patient sample⁵⁴.

13. Symbols



Use By



Catalog Number



Batch Code



In Vitro Diagnostic
Medical Device



Consult Instructions
for Use



Manufacturer



Do Not Reuse




X Number of Test
Devices In Kit



Manufacturing
Sequence



Serial Number

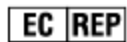

Authorized
Representative
in the European
Community



Temperature
Limitation



PN:
Part Number

Caution



denotes conformity to specified
European directives

14. Bibliography

1. Kramer B, et al. A simple technique for the determination of calcium and magnesium in small amounts of serum. *J Biol Chem* 1921; 47 : 475-481.
2. Clark EP, et al. A study of the Tisdall method for the determination of blood serum calcium with suggested modification. *J Biol Chem* 1925; 63 : 461-464.
3. Katzman E, et al. The determination of serum calcium by titration with ceric sulfate. *J. Biol Chem* 1937; 118 : 539-544.
4. Cali, et al. A reference method for the determination of total calcium in serum. In : *Selected Methods of Clinical Chemistry, Vol 8.* Cooper GR, ed. Washington, DC : American Association for Clinical Chemistry. 1977 : 3 -8.
5. Kessler G, et al. An automated procedure for the simultaneous determination of calcium and phosphorus. *Clin Chem* 1964; 10 : 686-703.
6. Michaylova V, et al. Photometric determination of micro amounts of calcium with arsenazo III. *Anal Chim Acta* 1971; 53 : 194-198.
7. Scarpa A, et al. Metallochromic indicators of ionized calcium. *Ann NY Acad Sci* 1978; 307 : 86-112.
8. Ono T, et al. A new enzymatic assay of chloride in serum. *Clin Chem* 1988; 34 : 552 -3.
9. Knoll VE, et al. Spezifische Kreatininbestimmung Im Serum. *Z Klin Chemi Clin Biochem.* 1970; 8 : 582-587.
10. Haeckel R, et al. Simplified Determinations of the "True" Creatinine Concentration In Serum And Urine. *J Clin Chem Clin Biochem.* 1980; 18 : 385-394.
11. Moss GA, et al. Kinetic Enzymatic Method For Determining Serum Creatinine. 1975; 21 : 1422 -1426.
12. Jaynes PK, et al. An Enzymatic, Reaction-Rate Assay For Serum Creatinine With a Centrifugal Analyzer. 1982; 28 : 114-117.
13. Fossati P, et al. Enzymatic Creatinine Assay : A New Colorimetric Method Based on Hydrogen Peroxide Measurement. 1983; 29 : 1494-1496.
14. Whelton A, et al. Nitrogen Metabolites and Renal Function. In : *Tietz Textbook of Clinical Chemistry, 2nd Ed.* Burtis CA, Ashwood ER, eds. Philadelphia : W.B. Saunders Company. 1994 : 1513 -1575.
15. Folin O, et al. A system of blood analysis. *J Biol Chem.* 1919; 38 : 81-110.
16. Somogyi M. A reagent for the copper-iodometric determination of very small amounts of sugar. *J Biol Chem.* 1937; 117 : 771-776.
17. Nelson N, et al. A photometric adaption of the Somogyi method for the determination of glucose. *J Biol.* 1944; 153 : 375-380.
18. Kaplan LA. Glucose. In : *Clinical Chemistry : Theory, Analysis, and Correlation, 2nd Ed.* LA Kaplan and AJ Pesce, eds. St. Louis : The C.V. Mosby Company. 1989 : 850-856.
19. Wacker WEC, Ulmer DD, Vallee BL. Metalloenzymes and myocardial infarction. *New England journal of medicine,* 1956; 225 : 449-3.
20. Tabata M, et al. Direct Spectrophotometry of magnesium in serum after reaction with hexokinase and glucose-6-phosphate dehydrogenase. *Clin Chem* 1985; 31 : 703 -5.
21. Berry MN, et al. Enzymatic determination of potassium in serum. *Clin Chem* 1989; 35 : 817-20.
22. Van Pelt J. Enzymatic determination of sodium, potassium and chloride in serum compared with determination by flame photometry, coulometry and ion selective electrodes. *Clin Chem* 1994; 40 : 846 -7.
23. Hubl W, et al. Enzymatic determination of sodium, potassium and chloride in abnormal (hemolyzed, icteric, lipemic, paraproteinemic, or uremic) serum samples compared with indirect determination with ion selective electrodes. *Clin Chem* 1994; 40 : 1528-31.
24. Helgerson RC, et al. Host-guest Complexation. 50. Potassium and sodium ion-selective chromogenic ionophores. *J Amer Chem Soc* 1989; 111 : 6339-50.
25. Kumar A, et al. Chromogenic ionophore-based methods for spectrophotometric assay of sodium and potassium in serum and plasma. *Clin Chem* 1988; 34 : 1709-12.
26. Berry MN, et al. Enzymatic determination of sodium in serum. *Clin Chem* 1988; 34 : 2295 -8.
27. Skeggs LT Jr. An automatic method for the determination of carbon dioxide in blood plasma. *Am J. Clin Pathol* 1960; 33 : 181-5.
28. Korzun WJ, Miller WG. Carbon Dioxide. In : *Clinical chemistry theory, analysis and correlation, 2nd Ed.* Kaplan LA, Pesce AJ, eds. St. Louis : The CV Mosby Company. 1989 : 869 -72.
29. Fales FW. Urea in serum, direct diacetyl monoxide method. In : *Selected Methods of Clinical Chemistry, Vol 9.* Faulkner WR, Meites S, eds., Washington, DC. : American Association for Clinical Chemistry. 1982 : 365 -373.
30. Van Slyke, et al. A permanent preparation of urease, and its use in the determination of urea. *J Biol Chem,* 1914; 19 : 211-228.
31. Fawcett JK, et al. A rapid and Precise method for the determination of urea. *J Clin Pathol,* 1960; 13 : 156 -159.
32. Chaney, et al. Urea and ammonia determinations. *Clin Chem,* 1962; 8 : 130 -132.
33. Talke H, et al. Enzymatische Harnstoffbestimmung in Blut and Serum im optischen Test nach Warburg. *Klin Wochensch,* 1965; 43 : 174-175.

34. Hallett, et al. Reduced nicotinamide adenine dinucleotide-coupled reaction for emergency blood urea estimation. *Clin Chim Acta*. 1971; 35 : 33-37.
35. Patton, et al. Spectrophotometric and kinetics investigation of the Berthelot reaction for the determination of ammonia. *Anal Chem*. 1977; 49 : 464-469.
36. Sampson EJ, et al. A coupled-enzyme equilibrium method for the measuring urea in serum : optimization and evaluation of the AACC study group on Urea Candidate reference method. *Clin Chem*. 1980; 26 : 816-826.
37. Schembri CT, et al. Centrifugation and capillarity integrated into a multiple analyte whole blood analyser. *J Automatic Chem* 1995; 17 : 99-104. (journal's name changed in 2000 to *J Automated Methods & Management in Chemistry*).
38. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). Physician's office laboratory guidelines, tentative guideline, 2nd ed. CLSI Document POL1-T2. Wayne, PA : CLSI, 1992.
39. Scott, M.G. Electrolytes and Blood Gases. In : *Tietz Textbook of Clinical Chemistry*. 3rd Ed. Burtis CA, Ashwood ER, eds. Philadelphia : W B Saunders Company. 1999 : 1058-9.
40. Scott, M.G. Electrolytes and Blood Gases. In : *Tietz Textbook of Clinical Chemistry*. 3rd Ed. Burtis CA, Ashwood ER, eds. Philadelphia : W B Saunders Company. 1999 : 1065-6.
41. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). Interference testing in clinical chemistry; proposed guideline. CLSI Document EP7-A. Wayne, PA : CLSI, 1986.
42. Young DS. Effects of drugs on clinical laboratory tests, 3rd Ed. Washington, DC : AACC Press, 1990.
43. Benet LZ, Williams RI. Design and optimization of dosage regimens : pharmacokinetic data. In : *Goodman And Gilman's the pharmacological basis of therapeutics*, 8th Ed. Gilman AG, et al., eds. New York : Mcgraw-Hill, Inc., 1990 : 1650-735.
44. Moss DW, Henderson AR. Clinical enzymology. In : *Tietz Textbook of Clinical Chemistry*. 3rd Ed. Burtis CA, Ashwood ER, eds. Philadelphia : W B Saunders Company. 1999 : 617-721.
45. Painter PC, Cope JY, Smith JI. Reference Information for the clinical laboratory. In : *Tietz Textbook of Clinical Chemistry*. 3rd Ed. Burtis CA, Ashwood ER, eds. Philadelphia : W B Saunders Company. 1999 : 1788 -1846.
46. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). How to define and determine reference intervals in the clinical laboratory, approved guidelines, 2nd ed. CLSI Document C28-A2. Wayne, PA : CLSI, 2000.
47. Bowers GN. Lactic dehydrogenase. In : *Standard Methods of Clinical Chemistry*, Vol 4. Seligson D, ed. New York : Academic Press. 1963 : 163-172.
48. Siest G, et al. Plasma enzymes—physiological and environmental variations. In : *Reference Values in Human Chemistry*. Siest G, ed. New York : Karger. 1973 : 28-38.
49. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). Evaluation of precision performance of clinical chemistry devices; approved guideline. CLSI Document EP5-A2. Wayne, PA : CLSI, 1999.
50. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). Quality management for unit-use testing; proposed guideline. CLSI Document EP18-A. Wayne, PA : CLSI, 1999.
51. Clinical and Laboratory Standards Institute (formerly, National Committee for Clinical Laboratory Standards,). Method comparison and bias estimation using patient samples; approved guideline. CLSI Document EP9-A2. Wayne, PA : CLSI, 1995.
52. CLSI GP44-A4 2010:Procedures for the Handling and Processing of Blood Specimens for Common Laboratory Tests; Approved Guideline—Fourth Edition
53. Overfield CV, et al. Glycolysis: a re-evaluation of the effect on blood glucose. *Clin Chim Acta*. 1972; 39: 35-40.
54. Hartland AJ, Hartland JH, Neary RN. Serum Potassium Is Unreliable as an Estimate of in Vivo Plasma Potassium. *Clin Chem* 1999;45:1091-1092.
55. CLSI GP41 7ED 2017: Collection of Diagnostic Venous Blood Specimens